


Somno-Art Portal Healthcare Professional Manual

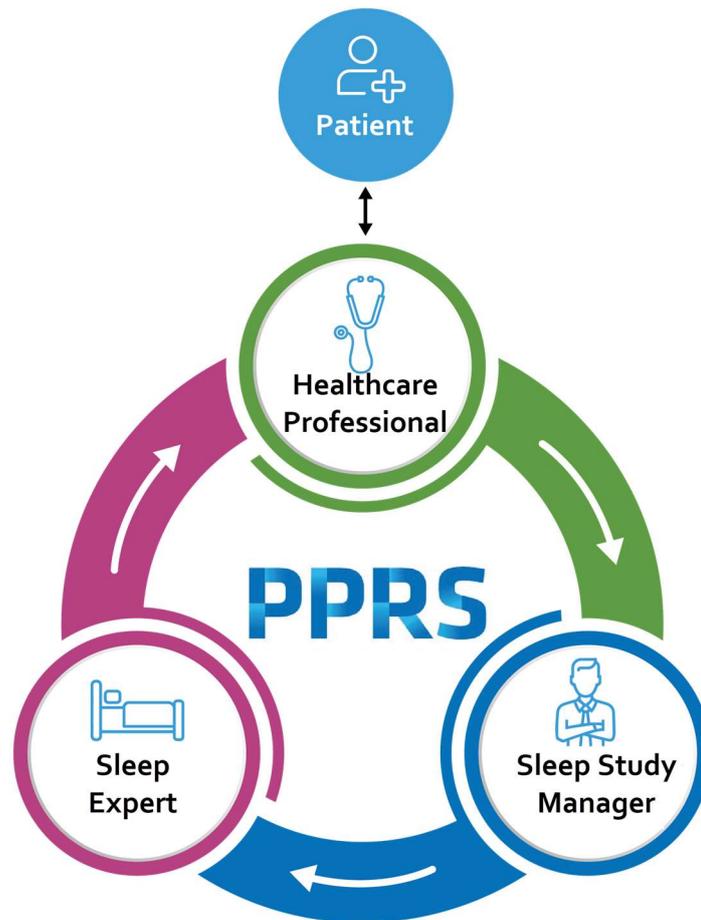

Contents

1	Introduction to the Data Flow of the PPRS Sleep Monitoring Device.....	3
1.1	Portals	4
1.2	Data Flow and Personnel	4
2	Healthcare Professional in the Device Data Flow	6
2.1	The Healthcare Professional	6

2.2	Giving the Device to the Patient.....	6
2.3	Uploading Device Data to the Portal.....	8
2.4	Receiving the Sleep Report	10

- **Sleep Expert** – The PPRS associate that verifies that the final Sleep Report is accurate and complete, so it can be passed to the Healthcare Professional

This user guide describes how **Healthcare Professional** use the **Somno-Art Portal** (graphical interfaces) to do their parts in the Somno-Art System Data Flow to serve patients that use the Somno-Art Device to gather data about their sleep patterns.

Somno-Art Device and its Companion is a CE Class IIa medical device.

Somno-Art Software is a CE Class I medical device.


Their respective Healthcare Professional manuals are available on <https://www.somno.art/>

1.1 Portal

- **Somno-Art Portal** – The graphical interface used by medical care organization personnel, for Healthcare Professionals.
<https://portal.somno.art/>

1.2 Data Flow

This diagram shows the Somno-Art System Data Flow (also referred to as “the Flow”) starting with the file created by the Somno-Art Device, through initial verification of the data quality, continuing with the final verification of the Sleep Report that the Healthcare Professional uses to continue the medical care of the patient.

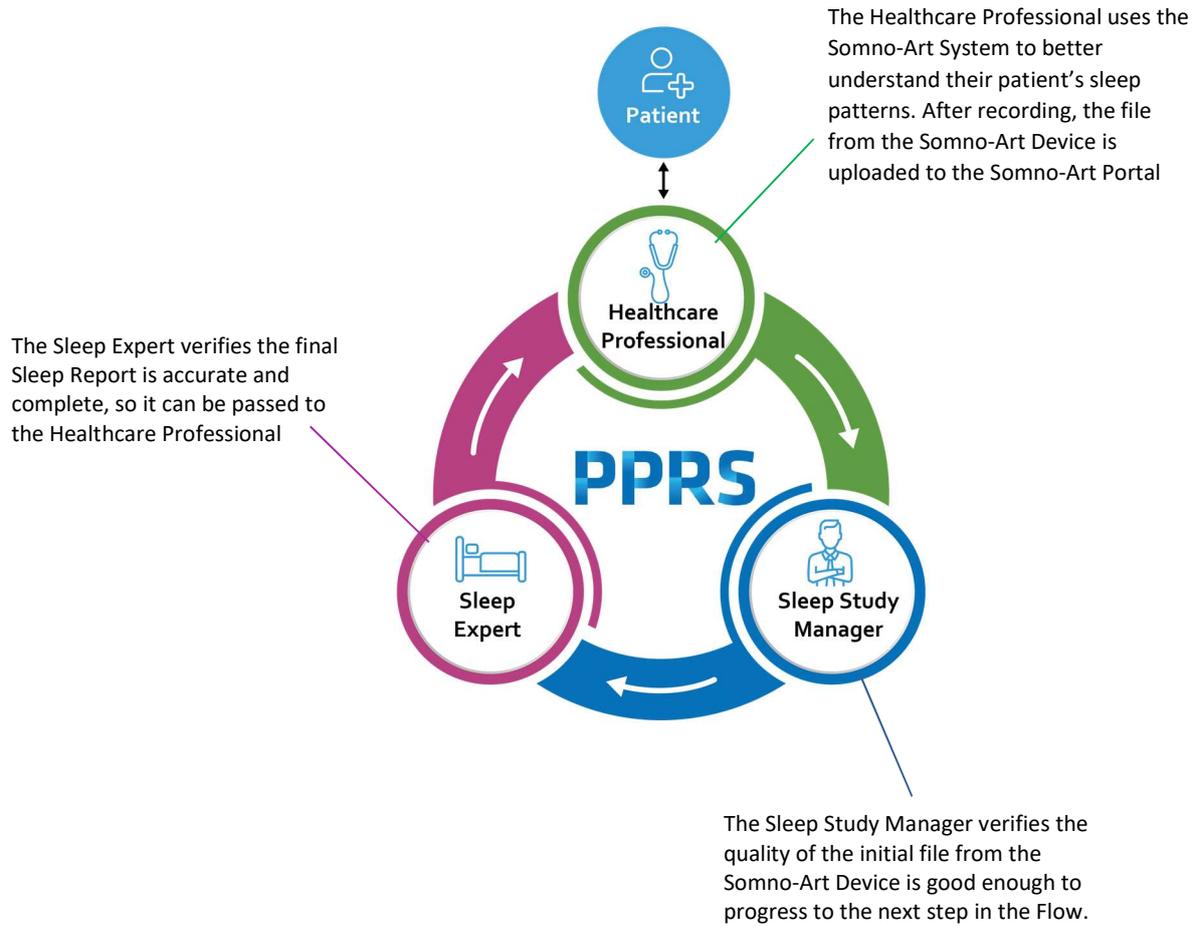

2 Healthcare Professional in the Device Data Flow

2.1 The Healthcare Professional

The Healthcare Professional is a medical care provider that interacts with patients and may use the Somno-Art System to better understand the patient’s sleep patterns.

The Healthcare Professional provides the Somno-Art Device to the patient to gather data about the patient’s sleep patterns. When the patient returns the Somno-Art Device, the Healthcare Professional will be able to import the data on his/her computer using the Companion software (provided with the Somno-Art Device), and then upload the data to the Somno-Art Portal.

The data is then processed as it goes through the steps of the Flow (see Data Flow and Personnel). When the data completes all steps of the Flow successfully, the Healthcare Professional receives a Sleep Report that can be used for medical decisions.

2.2 Adding a Patient

If patient information have not been entered before, you can add them to the Somno-Art Portal as follows: Access the Somno-Art Portal at <https://portal.somno.art/>

2. Under **Patients**, select **Add+**.

Name	Patient Code	Date of Birth	Gender
Dubois Richard	P017	9/13/22	Male
Karswell Simon	P014	9/4/22	Male
Bernard Sophia	P011	9/2/22	Female
Durand Mike	P001	9/9/22	Male

Enter the details for the new patient and then press **Add**.

Add Patient (step 1/1) x

STEP 1 - GENERAL

Name*

Laurent Pocant

Gender* Male **Date of Birth*** 10/02/1965

Description

Type

Phone Type a phone **Additional Phone** Type a phone

Address

Country State

Add

The patient is added to the list of patients.

Note: You can see the patient details by clicking on the patient in the list and then clicking **Expand**. You can then edit the details by clicking **Edit Details**. You can delete a patient by clicking **Delete** located in the top right corner of the selected patient.

Organization

Dubois Richard | Patient

Overview Observations Sleep Session

Patients

4 Items [Add +](#)

Name ▾


- Dubois Richard [Edit Details](#)
- Karswell Simon
- Bernard Sophia
- Durand Mike

Name	Dubois Richard	Gender	Male	Date of Birth	9/13/22
Description		Phone		Additional Phone	
Address		Creation time	9/21/22, 6:10 PM	Clinic	NY_Med ↗
Caregiver	Pepsi Grepsi ↗	National ID	12389uhe	Last modified time	10/1/22, 4:28 PM
Device		Sleep Session		Email	


2.3 Uploading Device Data to the Portal

When the patient returns the Somno-Art Device, you need to upload the data from the device to the Somno-Art Portal, as follows:

1. Access the Somno-Art Portal at <https://portal.somno.art>
2. Import the SAR file from the Somno-Art Device to the computer using the Companion.
3. Click on the name of the patient, and then click on **Expand**.
4. Click on **Sleep Sessions** to see all of the sleep sessions for the patient.

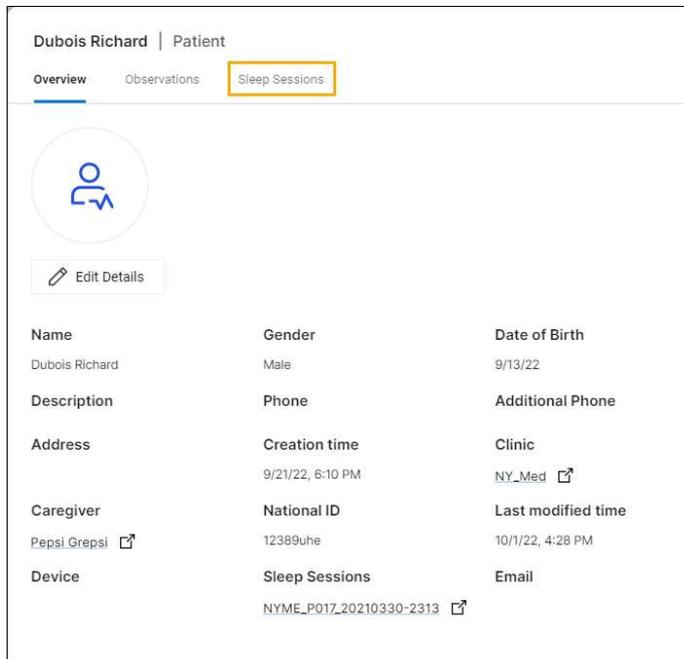


5. Click **+ Sleep Session** and then click **Add new**.

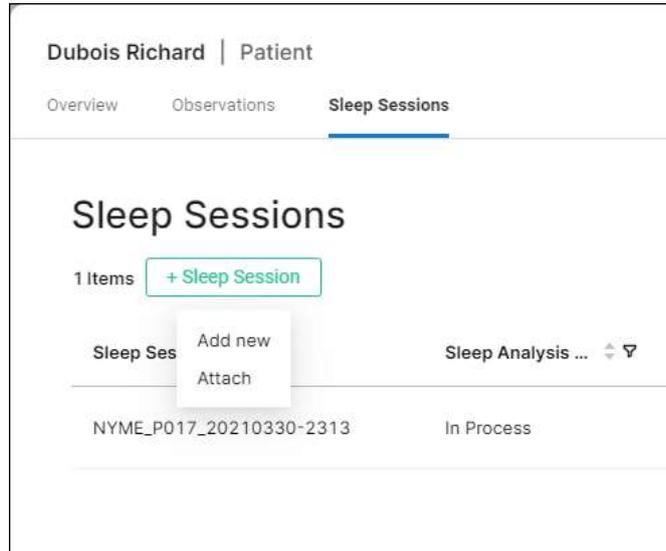

6. Select the Somno-Art Device Serial Number used by the patient.

7. Upload the SAR file.

Click **Add**.

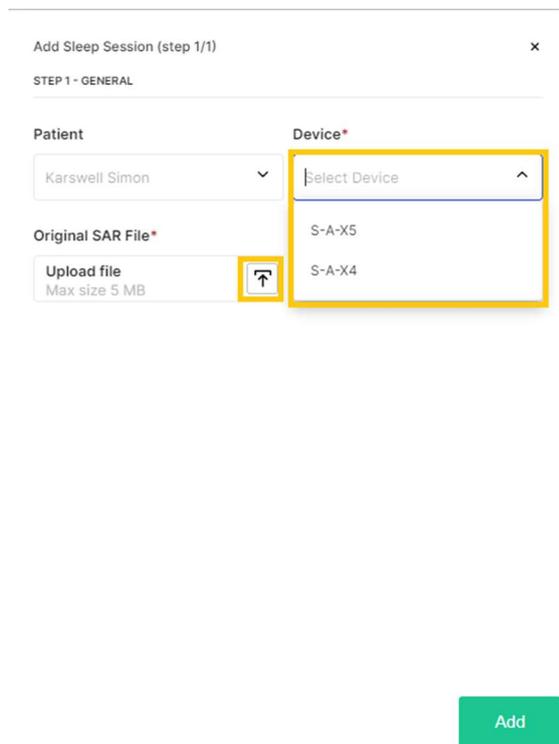

2.4 Receiving the Sleep Report

1. Access the Somno-Art Portal at <https://portal.somno.art/> and select **Patients**.

Note: If you received an email notification, the link will take you to the **Patients** page.

2. Find and select a patient.
3. Click **Expand**.

The screenshot displays the 'Patients' section of the Somno-Art Portal. At the top, there are navigation tabs for 'Organization', 'Patients', 'Caregivers', and 'Organization Users'. The 'Patients' tab is active. Below the navigation, there is a search bar and a 'Delete' button. The main content area shows a list of 4 patients. The first patient, Dubois Richard, is highlighted. To the right of the list, a detailed view for Dubois Richard is shown, including a profile picture, name, patient status, and an 'Expand' button. The 'Expand' button is highlighted with a yellow box.

Name	Patient Code	Date of Birth	Gender
Dubois Richard	P017	9/13/22	Male
Karswell Simon	P014	9/4/22	Male
Bernard Sophia	P011	9/2/22	Female
Durand Mike	P001	9/9/22	Male

Dubois Richard
Patient

Expand

Name Dubois Richard
Date of Birth 9/13/22
Gender Male
Phone
Email

4. Click **Sleep Sessions**.

Dubois Richard | Patient

Overview Observations **Sleep Sessions**



 Edit Details

Name	Gender	Date of Birth
Dubois Richard	Male	9/13/22
Description	Phone	Additional Phone
Address	Creation time	Clinic
Caregiver	National ID	Last modified time
Device	Sleep Sessions	Email

5. Select a Sleep Session with status **Analysis Ready**.

Durand Mike | Patient Disable

Overview Observations **Sleep Sessions**

Sleep Sessions Search Delete 

2 Items [+ Sleep Session](#)

Sleep Session ID ▾	Sleep Analysis Status ▾	Creation time ▾	Last modified time ▾	
Mike_P001_20210330-2313	Analysis Ready	9/20/22, 9:40 AM	9/20/22, 9:56 AM	⋮
d978_P001_20210330-2313	In Process	9/16/22, 11:19 PM	9/21/22, 10:00 AM	

Mike_P001_20210330-2313
Sleep Session

[Expand](#)

Sleep Session ID
Mike_P001_20210330-2313

Sleep Analysis Status Analysis Ready

Sleep Report
 [9f8b0f45-1715-48f7-b8b1-0f0543a4e7b2 Sleep_Report.pdf \(858.5 KB\)](#)

Healthcare Professional Feedback
Hi Mike

< 1 > 1-2 of 2

6. On the right side, click **Sleep Report** to download the PDF file.

7. Repeat this for all sleep sessions with the status **Analysis Ready** under **Sleep Analysis Status**.

Note: You can see the Sleep Session by clicking **Expand**.

The Sleep reports can also be downloaded in the Sleep Sessions tab, by clicking directly the download button in the list